

Keto-Mojo® LANCING DEVICE

PREPARATION & USE

The lancing device comes with every GK+ Blood Glucose & Ketone Meter Kit and is used to draw blood from the skin.

NOTE: Always wash your hands before you test or use an alcohol swab to sterilize the test site.



Twist cap to remove.

Always use a sterile lancet every time you test.



PRESS THIS END TO YOUR SKIN

The needle of the lancet fires from the tip of the cap and immediately retracts.

LANCET DEPTH ADJUSTER

Turn the tip of the lancet cap to find your preferred needle depth.

Shallow 1 2 3 4 5 Deep



INSTALLING THE LANCET

Open the cap and insert the lancet into the expansion gap. Press firmly as the fit is designed to be tight. Replace the cap and twist to lock.

NOTE: The Keto-Mojo lancing device supports most generic lancets.

EXPANSION GAP



ACTIVATION BUTTON

Place the tip of the lancet device on your desired testing location and press the button when you are ready.

PULL KNOB TO ACTIVATE

Pull the gray knob at the bottom of the lancing device to activate. When you pull the knob, your lancing device is ready to use.



We suggest using the side of your finger to test. If you are having trouble drawing blood, squeeze the base of your finger and slide up towards the tip of your finger.

For more tips, visit keto-mojo.com.